
SAFEWAY 500 SNABBGUIDE

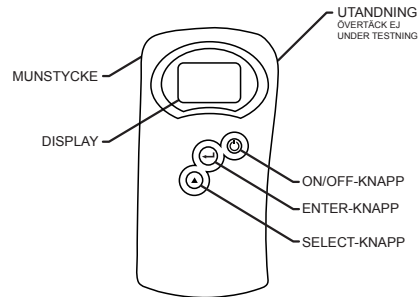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
SAFEWAY 500 QUICK GUIDE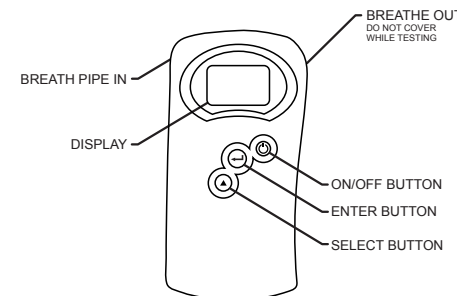

To toggle through available menu options, press the SELECT ⏸ button. See manual for details on the menu options.

THIS DEVICE IS INTENDED TO MEASURE ALCOHOL IN THE HUMAN BREATH.

DO NOT DRINK AND DRIVE.

SEE OWNER'S MANUAL FOR RE-CALIBRATION INSTRUCTIONS THAT MUST BE ROUTINELY PERFORMED.

Be sure to read the entire owner's manual and follow procedures carefully to obtain optimal results.

Operating Instructions

Wait 20 minutes after the test subject smokes, eats or drinks before testing. Failure to wait 20 minutes can give inaccurate readings and damage the sensor.

Step 1) Press and hold the ON/OFF power button ⏻ for one (1) second until you hear a BEEP. The unit reads "Initializing..." as it prepares for use.

Step 2) The screen displays "Test" and "Options". To start a "Test", press the ENTER button ⏻. The unit displays "Take a Deep Breath" for three (3) seconds, beeps, and then displays "Blow Now". Take a deep breath, wrap your lips around the new mouthpiece and blow steadily for five (5) seconds. The unit takes the breath sample in the amount of time set for a breath sample (factory default is 5 seconds). Do not cover the BREATH OUT hole.

Note: Begin blowing immediately when "Blow Now" is displayed to get a deep lung sample.

Step 3) The unit displays "Processing" while analyzing the sample. After about three (3) seconds, the unit then displays the estimated blood alcohol level.

Step 4) The unit shows the test result for 90 seconds before beeping twice and shutting off.

Step 5) To turn off the unit, press and hold the ON/OFF power button ⏻ for one (1) second.